

Product: Panther Unloader Knee Brace

FEATURES:

- Extremely light-weight
- Low-profile brace provides 3 points of leverage to reduce pressure inside the knee joint
- Ideal for unilateral compartment OA Knee users
- Valgus and Varus adjustment to fit the patient's unique alignment

INDICATIONS:

Ideal for patients with mild to moderate osteoarthritis of the knee.



| Order # | Location |
|---------|------------------------------------|
| 950-R | Fits Right Lateral and Left Medial |
| 950-L | Fits Left Lateral and Right Medial |

HOW TO APPLY: Open all straps and step through the brace. With leg slightly bent, center hinge one (1) inch above the top of the knee cap. Push hinge slightly back behind the midline of the knee.

Attach the first strap label on the number system shown on the straps. Pull the strap on top of the knee area, making sure it is snug.

Attach all the straps on the buckle, making sure it is snug. Stand up and walk, making sure it is not too tight or too loose.

CARE INSTRUCTIONS: Hand wash in cold water with mild soap. Air dry away from heat. Don't tumble dry. Don't bleach.

CAUTION: Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

SUGGESTED HCPCS CODE: L1843 and L1851

